Quitamet Ultra

EN DANGER

DE GEFAHRLICH

NL GEVAAR

ES PELIGRO

PT PERIGO

IT PERICOLO

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U_L62567P20-150209

OPEN

EN**Cleaning and disinfection of dental instruments**

Composition: Didecyldimethylammonium chloride, chlorhexidine digluconate, non-ionic surfactants, fragrance, colouring, excipients.

Microbiological properties: Bactericide in 5 min, 0,5%, 20°C (EN 1040, EN 13727, EN 14561). Active against MRSA in 5 min, 0.5%, 20°C (EN 13727). Tuberculocide in 60 min, 2%, 20°C (EN 14348, EN 14563). Yeasticide in 5 min, 2%, 20°C (EN 1275, EN 13624, EN 14562). Active in 5 min, 0.5%, 20°C against HIV-1 PRV (surrogate of HBV), BVDV (surrogate of HCV), Vaccinia virus, Herpesvirus.

Instructions for use: Concentrated solution for professional use. Use for soaking at a dilution of 2%, i.e. 20 ml for 1 litres of warm or cold water, respecting a contact time of 5 min. For tuberculocidal activity, dilute to 2%, respecting a contact time of 60min. Can be used in ultrasonic bins. Brush if necessary. Rinse thoroughly. Renew the soaking bath at least once a day.

Precautions of use: DANGER. Contains didecyldimethylammonium chloride and alkyl polyglycol ether C8-C10 with ethylene oxide. Causes serious eye damage. Causes skin irritation. Very toxic to aquatic life with long lasting effects. Use in well-ventilated areas. Wear protective gloves and protective eye/face equipment. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician. IF ON SKIN: Wash with plenty of water. If necessary, consult a doctor. Keep out of reach of children. Do not reuse the packaging. Avoid release to the environment. This material and its container must be disposed of as hazardous waste. For professional dental use only.

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