FR

UTILISATION

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EN

INTENDED USE

GUTTASOLV is a solvent for gutta-percha.

COMPOSITION

Eucalyptol.

INDICATIONS

- Fitting of the primary gutta-percha point in the root-canal, using the lateral condensation method.
- Retreatment after root canal filling with gutta-percha.

GUTTASOLV is indicated for mature and permanent teeth.

PERFORMANCES

GUTTASOLV dissolves or softens gutta-percha used in endodontic treatments.

CONTRA-INDICATIONS

Hypersensitivity to Eucalyptol.

INSTRUCTIONS FOR USE

The operative field shall be completely isolated: the use of a rubber dam around the tooth is recommended.

FITTING OF THE PRIMARY POINT (LATERAL CONDENSATION):

- Select the primary gutta-percha point,
- Dip the apical tip into the solvent,
- Insert the tip into the canal and proceed as usual.

RETREATMENT:

- Place two drops of solvent into the pulp chamber,
- Remove the softened gutta-percha using a manual root canal instrument,
- Add some fresh solvent from time to time and rinse alternately with sodium hypochlorite.

ADVERSE EFFECTS

May cause an allergic reaction.

WARNINGS AND PRECAUTIONS OF USE

Close the cap securely after each use.

Warning

Flammable liquid and vapour.

Do not allow the product to enter sewers/ surface water or ground water. Refer to safety data sheet - available upon request.

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